<b>3M ESPE</b>
<b>Durelon™</b>
<b>Carboxylate Cement</b>
<b>Carboxylatzement</b>
<b>Ciment au carboxylate</b>
<b>Cemento carbossilico</b>
<b>Cemento de carboxilato</b>
<b>Cimento carboxilado</b>
<b>Carboxylaatcement</b>
<b>Πολυκαρβοξυλική κοινά</b>
<b>Karboxylatzement</b>
<b>Karboksylaattipohjainen sementti</b>
<b>Carboxylatzement</b>
<b>Karboksylatsement</b>

Instructions for Use
Gebrauchsinformation
Mode d'emploi
Istruzioni per l'uso
Información de uso
Instruções de uso

**3M ESPE AG**

Dental Products

D-82229 Seefeld - Germany

**3M ESPE**

Dental Products

St. Paul, MN 55144-1000

3M ESPE Technical Hotline/MSDS Information in U.S.A. and Canada 1-800-634-2249.

3M, ESPE, Alkaliner and Durelon are trademarks of 3M or 3M ESPE AG.

© 3M 2003. All rights reserved.

44000273693/01

## ENGLISH

**Product Description**
Durelon, manufactured by 3M ESPE, is a carboxylate cement for fixation and lining. Two variants of the powder, "normal curing" and "fast curing", and two consistencies of the liquid, "universal" and "low viscosity", are available. Two delivery versions of the "universal" liquid, dropper bottle and dosage syringe, are available.The "low viscosity liquid" is available with a dropper bottle only. Either of the powder variants and the "universal" liquid may be used for lining. Any of the combinations can be employed for fixation. These instructions for Use should not be discarded for the duration of product use.

### Fields of Application

- Fixation of metal, metallo-porcelain or composite veneered inlays, onlays, crowns and bridges
- Fixation of composite or ceramic inlays, onlays, crowns and bridges, if suited for conventional cementing
- Fixation of rods and screws, if suited for conventional cementing
- Fixation of orthodontic bands
- Linings

Durelon is not suitable for cementing titanium frameworks.

### Preparation

To provide for optimal adhesion, carefully clean and dry enamel, dentin, and metal surfaces. Residues of any kind reduce adhesion. Do not overdry!

### Pulp protection

To prevent pulp irritation, areas in the vicinity of the pulp should be dabbed with hard-curing calcium hydroxide preparation (e. g. Alkaliner™, manufactured by 3M ESPE) **before** taking an impression of the tooth for the definitive restoration.

### Dosing

Powder: Assemble the dosing rod prior to use! Press the dosing rod into the powder, and then remove excess powder with some instrument. Dose the powder onto a mixing block or glass plate by lightly tapping the dosing rod.

Avoid under-dosing the powder (especially in free dosing), since this may impair the quality of the cement.

Liquid: Holding the bottle vertically dispense single droplets next to the powder by lightly squeezing the bottle.

If you wish to use the dosing syringe, insert the enclosed plunger and dose the desired number of scale units. Keep the plunger for subsequent use with other syringes. The bottle or dosing syringe should be resealed immediately after use.

Mixing ratio
1 dose powder = 1 dosing rod filling,
1 dose liquid = 1 drop from the bottle or 1 small scale unit from the dosing syringe.

Indication	Powder		Liquid	
	normal setting	fast setting	universal	low viscosity
Number of doses for fixation	1		2	
	1	1	2	1
Number of doses for linings	1		1	
	1	1	1	

The following powder/liquid mixing ratios apply:
for fixation: 0.15 g powder: 0.10 g liquid universal or 0.20 g powder: 0.10 g liquid low viscosity,
for lining: 0.25 g powder: 0.10 g liquid universal
These quantities are not identical to the quantities required for one application!

### Mixing

Process Durelon at an ambient temperature of 20–25°C/68–77°F. Mix with a metal or plastic cement spatula. Place all powder into the liquid in one move and mix until a homogeneous consistency is attained.

### Application

The material adheres to metal instruments and, hence, should be washed off with cold water prior to setting.

Keep the work area dry during application of the material. Neighboring teeth or crowns may be lubricated with vaseline. Apply a thin layer of cement to the inside of the crown and the prepared tooth. Avoid overfilling the crown. Subsequently, insert the restoration.

**Times**
The following times are valid at an ambient temperature of 23°C/73°F and 50% relative humidity:

	Mixing	Processing incl. mixing	Setting from start of mixing
	min <span> </span> : <span> </span> sec	min <span> </span> : <span> </span> sec	min <span> </span> : <span> </span> sec
Fixation:			
Powder, normal setting Liquid, universal	00 <span> </span> : <span> </span> 30	02 <span> </span> : <span> </span> 30	10 <span> </span> : <span> </span> 00
Powder, normal setting Liquid, low viscosity	00 <span> </span> : <span> </span> 30	02 <span> </span> : <span> </span> 00	07 <span> </span> : <span> </span> 00
Powder, fast setting Liquid, universal	00 <span> </span> : <span> </span> 30	01 <span> </span> : <span> </span> 00	05 <span> </span> : <span> </span> 00
Lining:			
Powder, normal setting Liquid, universal	00 <span> </span> : <span> </span> 30	01 <span> </span> : <span> </span> 30	06 <span> </span> : <span> </span> 00
Powder, fast setting Liquid, universal	00 <span> </span> : <span> </span> 30	00 <span> </span> : <span> </span> 45	03 <span> </span> : <span> </span> 30

Processing times are shortened at higher temperatures, and prolonged at lower temperatures than those stated (e. g. mixing on a chilled glass plate). Using a larger quantity of powder also shortens the processing time. Exceeding these processing times (apparent by the matte surface of the paste) leads to a loss of adhesion to enamel and dentin.

### Removing Excess Material

To simplify cleaning, remove any excess material shortly before definitive setting of the material using an instrument. Be careful to avoid pulling any material from the seam. After the setting time has elapsed remove any remaining excess material, e. g. with a probe.

### Notes

Dose the liquid directly prior to mixing to avoid the evaporation of ingredients. Powder and liquid bottles and the dosing syringe should be carefully resealed after use.

### Incompatibilities

In susceptible individuals, sensitization to the product cannot be excluded. Use of the product should be discontinued and the product completely withdrawn from contact with the patient, if allergic reactions are observed. In rare cases, complications of the pulp may occur, especially if the recommendations for processing were disregarded.

### Storage and Shelf-life

Do not store the product above 25°C/77°F. Do not use after the expiration date.

### Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

### Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

### Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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